

Allflex EID-Large Visual Matched Sets



General Description

The Allflex EID-Large Visual Matched sets are a combination of HDX High Performance Ultra EID tag with a matching Large Visual tag with corresponding laser/ink printed numbering. The full EID number is printed in the neck area of the visual tag. All tags feature Tamperproof design.

The Allflex package of EID tags include a bag code. This bag code can be entered in the Allflex EID Tag Number Look Up site (www.allflexlink.com), and the numbers contained in the package can be downloaded. All bags contain a User Guide for the Website. This information can be easily migrated or merged to an existing database or management program without any reading equipment.

Options

The EID-Large Visual Matched sets are available in three customizable combinations, which include basic tags with a management number, to combinations with logos and reverse-side marked text. Available in seven colors.

Packaging

The Matched Sets are packaging in a plastic tray in sets of five tags. The standard quantity per bag is 25 matched sets.

Ordering Information

SKU

EID/GTLF-COLOR-SET1

EID/GTLF-COLOR-SET2

EID/GTLF-COLOR-SET3



Colors available: Blue, Green, Orange, Red, White, Pink, and Yellow

Set 1



Set 2



Set 3



Allflex USA, Inc.

P.O. Box 612266 * Dallas/Fort Worth Airport, TX 75261-2266
800-989-TAGS (8247) * Customer Toll Free FAX 877-456-3969


Allflex Canada

4135 Berard * St. Hyacinthe, Quebec J2S 8Z8
866-505-TAGS (8247) * FAX 450-261-8028

www.allflexusa.com

Allflex EID-Maxi Visual Matched Sets

Packaging Example of Matched Sets in Nested Trays
Perforated for easy detachment for individual use

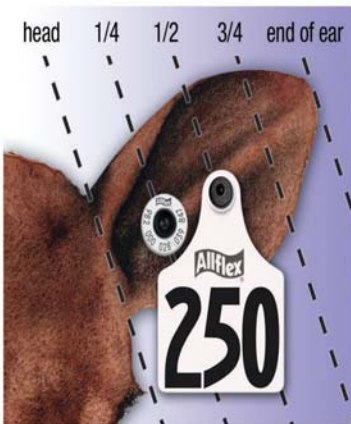

Recommended EID Tag Placement for Cattle

Application site must be free of foreign debris prior to placement of tags on the animal. Review application instruction prior to tagging.

1. The EID tag should be placed in the middle of the ear between the two cartilage ribs close to the head.
2. The female portion of the tag should be on the inside of the ear with EID tag application. Note that this is a thicker part of the ear. Application may be more difficult than when applying a visual tag.
3. **IMPORTANT:** Caution, "Free Air Space" is critical for proper healing and retention. Inspect placement after tagging to insure there is sufficient space between ear and EID tag.

