


# APPLICATION INSTRUCTIONS

## *Lightweight Sheep & Goat EID tag (vented cap).*

### Application Instructions for Allflex Electronic Identification Ear Tags

Apply Allflex vented cap Electronic ID Tags with the Red Universal Total Tagger. Use the red blunt pin and the **black insert** in the base of the jaws.

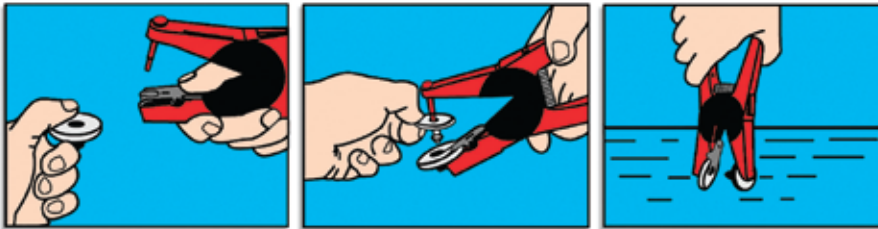

1. To load, depress spring clip and insert the female EID tag.
2. Slip the male tag completely onto the blunt applicator pin. Squeeze the jaws together lightly to ensure the male shaft is in line with the female.
3. Dip the jaws of the applicator holding the tag into an antiseptic or disinfectant solution.



800.989.TAGS • [allflexusa.com](http://allflexusa.com)

Allflex USA, Inc. • P.O. Box 612266 • Dallas/Fort Worth Airport, TX 75261-2266

